

HOW TO ADJUST THE ROAMI FOR CHANGING SURFACES

STAIRS & SLOPES

1

WHEELS UP

ONE TOUCH OF THE PEDAL AUTOMATICALLY RETRACTS THE WHEELS.



*RETRACTABLE WHEELS SOLD SEPARATELY

Full instructional video: www.Roamiwalkers.com



DO NOT ADJUST THE FOLD DIAL WHEN POSITIONING ROAMI. (DIAL IS FOR FOLDING/UNFOLDING ONLY)

2

PRESS & RELEASE SAFETY BUTTON

THE EASY-TO USE SAFETY BUTTON UNLOCKS THE ROAMI, ALLOWING IT TO BE ADJUSTED TO THE DESIRED ALIGNMENT.

HOW TO ADJUST THE ROAMI FOR CHANGING SURFACES

STAIRS & SLOPES

3

PLACE ROAMI ON SURFACE



Full instructional video: www.Roamiwalkers.com



4

SQUEEZE LEVER & ADJUST ROAMI TO SURFACE

RELEASE LEVER TO LOCK INTO POSITION